306CH EXTERNAL BATTERY CHARGER

EN DeVilbiss iGo® Portable Oxygen System External Battery Charger

Made in USA. Instruction Guide.

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IEC SYMBOLS

	Attention - Consult Instruction Guide		Electric Shock Hazard
	Electrostatic-Sensitive Device		Consult Operating Instructions
	Class II electrical protection-double insulated		It is mandatory to read Operating Instructions prior to use
	External Power Present Indicator (solid light). Power Fail Alarm (flashing light)		Do not use in bathtub, shower, or water-filled reservoir
	Service Required		The device contains electrical and/or electronic equipment that must be recycled per EC Directive 2002/96/EC - Waste Electrical and Electronic Equipment (WEEE)

IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, basic safety precautions should always be followed. Read all instructions before using. Important information is highlighted by these terms:

WARNING Important safety information for hazards that might cause serious injury or death.

CAUTION Information for preventing damage to the product.

NOTE Information to which you should pay special attention.

Important safeguards are indicated throughout this guide; pay special attention to all safety information.

READ ALL INSTRUCTIONS BEFORE USING.

SAVE THESE INSTRUCTIONS

WARNING

- Electric Shock Hazard. Do not disassemble. The DeVilbiss iGo External Battery Charger contains no user-serviceable parts. If service is required, contact your DeVilbiss provider or authorized service center.
- Do not install, assemble or operate this equipment without first reading and understanding this instruction manual. If you are unable to understand the warnings, cautions and instructions, contact your provider or technical personnel before attempting to use this equipment – otherwise, injury or damage may occur.
- Keep all units away from children. Do not allow unauthorized or untrained individuals to operate the equipment.
- Never tamper with or try to repair the equipment yourself. If you have questions or suspect your equipment is not operating properly, contact your provider.
- If the charger has a damaged cord or plug, if it is not working properly, or if it has been dropped, damaged or submersed in water, do not use the unit and call a qualified technician for examination or repair.
- Do not use in temperatures greater than 40°C (104°F) or below 5°C (41°F).
- Do not use around tub, shower, or sink. Do not immerse in liquids or subject to harsh conditions.
- Keep the equipment at least five feet away from heat sources, electric or gas heaters of any kind, fireplaces, or stoves.
- Never attempt to lubricate the equipment.
- Never use aerosol sprays near the equipment.

CAUTION—Federal law (USA) restricts this device to sale by or on the order of a physician.

INTRODUCTION

Thank you for purchasing the DeVilbiss iGo External Battery Charger, model 306CH. Using multiple battery packs with your iGo Portable Oxygen System increases your ability to live a flexible and mobile lifestyle. The iGo battery charger allows you to charge spare battery packs at home while you are using the iGo concentrator.

The iGo battery charger should only be used to charge the rechargeable battery packs used in the DeVilbiss iGo Portable Oxygen System. Do not attempt to charge any other battery packs in the 306CH iGo battery charger.

INDICATIONS FOR USE

The model 306CH battery charger is used to charge the rechargeable battery pack used exclusively in the DeVilbiss iGo Portable Oxygen System.

IMPORTANT PARTS OF YOUR DEVILBISS IGO BATTERY CHARGER

- 1. Battery Slot
- 2. Charge Connectors
- 3. Control Panel
- 4. External Power Light (Green)
- 5. Service Required Light (Red)
- 6. Charge Indicator Light (Green)
- 7. External Power Input (on back)
- 8. Service Connection (on bottom) Factory Use Only
- 9. AC Power Cord (not shown)

OPERATING YOUR IGO BATTERY CHARGER

- 1. Place the iGo External Battery Charger on a flat, clean, dry location near a wall outlet.
- 2. Ensure that the charger is properly connected to an AC power source with adequate voltage (100-240VAC). While the charger is connected to an AC power source, the External Power Light will illuminate.
- 3. Insert a DeVilbiss iGo Battery Pack (part number 306D-413) into the charger with the front of the battery pack facing forward so that the power connectors on the back of the battery pack align with the charge connectors in the battery slot.
 - a. While the battery is being charged, the green Charge Indicator Light will flash.
 - b. When the battery is fully charged, the green Charge Indicator Light will stop flashing and change to a constant green light.
- 4. You may remove the battery pack at any point in the charge cycle.
- 5. The battery can remain in the charger after it is fully charged as long as the charger is powered.

CAUTION–Do not leave the battery in the charger if the charger is not powered. The charger will discharge the battery if the charger is not externally powered.

NOTE–If a faulty battery is inserted into the charger, it may take up to 30 seconds for the charger to indicate the fault.

NORMAL OPERATION

 Green Charge Light	 Red Service Light	Indication	 Green External Power Light
ON	ON	When power is first applied to the charger, both the green charge light and the red service light should illuminate no longer than 3 seconds.	ON- Receiving AC Power/ Normal Operation
FLASHING	OFF	Battery is charging	ON - AC Power/Normal Operation
ON	OFF	Battery is fully charged	ON - AC Power/Normal Operation

CHARGE INDICATOR LIGHTS/TROUBLESHOOTING

 Green Charge Light	 Red Service Light	 Green External Power Light	Indication	Reasons	Remedy
ON	ON	ON	Green Charge Light and Red Service Light remain lit after the normal 3-second test at start-up	Charger Error	Contact your DeVilbiss provider to repair / replace unit
OFF	OFF	ON	Charger does not detect an installed battery	1. Battery not inserted properly 2. Charger or battery error	1. Reinsert battery per instruction guide 2. Contact your DeVilbiss provider to repair / replace unit
OFF	ON	ON	Battery Error	Battery Error	Reinsert battery per instruction guide. If symptoms persist contact your DeVilbiss provider to repair / replace unit

 Green Charge Light	 Red Service Light	 Green External Power Light	Indication	Reasons	Remedy
OFF	FLASHES	ON	Battery failed voltage output test	Battery Error	Reinsert battery per instruction guide. If symptoms persist contact your DeVilbiss provider to repair / replace unit
FLASHES	FLASHES	ON	Battery does not charge	Charger's internal temperature is too hot to continue the charge cycle due to high ambient temperature or fan failure	1. Ensure adequate airflow is available around the charger 2. Ensure air vent holes are not occluded or blocked 3. Place charger in a cooler location for 1 hour. 4. If symptoms persist after 1 hour in cooler location, contact your DeVilbiss provider to repair / replace unit
FLASHES	ON	ON	Battery does not charge	Battery pack is too hot to continue the charge cycle due to high ambient temperature or battery failure	1. Remove battery pack from charger and place in a cool location for 30 minutes. 2. Reinsert the cooled battery 3. If symptoms persist a. insert a second battery, if one is available. If second battery resolves issue, replace failed battery b. contact your DeVilbiss provider to repair / replace unit
OFF	OFF	OFF	Charger does not detect AC power source	1. Charger is not receiving correct power 2. Power cord is not properly connected to the charger 3. Power cord or charger is faulty	1. Ensure the charger power cord is fully inserted into a live wall outlet 2. Check the power connection at the back of the charger 3. If problem persists after verifying power source, contact your DeVilbiss provider to repair / replace unit

CARING FOR YOUR IGO BATTERY CHARGER

WARNING

Before attempting any cleaning procedures, unplug the unit from the wall outlet.

Battery slot and air vents should be kept free of dirt or contaminants.

Use a soft, damp cloth to clean the surfaces of the unit.

SPECIFICATIONS

Dimensions.....	7.75" (19.69 cm) W x 4.75" (12.07 cm) D x 3.825" (9.72 cm) H
Input Power	100-240V~
Charge Time.....	Approximately 3 hours depending on ambient temperature and residual charge
IPX0 Rating	
Operating Temperature.....	5°C to 40°C (41°F to 104°F)
Electrical Requirements	100-240V~, 50/60 Hz
Maximum input current rating (100-120V~).....	1.9A (rms)
Maximum input current rating (220-240V~).....	1.1A (rms)
Maximum charging output	28V === 3.58A Max

Storage Conditions

Temperature:	-20°C to + 60°C (-4°F to 140°F)
Humidity.....	10% to 95% R.H., non-condensing
Barometric Pressure (tested at ~933 hPa).....	500 hPa to 1060 hPa
Mode of operation	Continuous
Equipment not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide	

GUIDANCE AND MANUFACTURER'S DECLARATION

WARNING

Medical Electrical Equipment needs special precautions regarding EMC and needs to be installed and put into service according to the Electromagnetic Compatibility [EMC] information provided in the accompanying documents.

Portable and Mobile RF Communications Equipment can affect Medical Electrical Equipment.

The equipment or system should not be used adjacent to or stacked with other equipment and that if adjacent or stacked use is necessary, the equipment or system should be observed to verify normal operation in the configuration in which it will be used.

NOTE– The EMC tables and other guidelines provide information to the customer or user that is essential in determining the suitability of the Equipment or System for the Electromagnetic Environment of use, and in managing the Electromagnetic Environment of use to permit the Equipment or System to perform its intended use without disturbing other Equipment and Systems or non-medical electrical equipment.

Guidance and Manufacturer's Declaration – Electromagnetic Emissions			
This device is intended for use in the electromagnetic environment specified below. The customer or user of this device should assure that it is used in such an environment.			
Emissions Test	Compliance	Electromagnetic Environment – Guidance	
RF Emissions CISPR 11	Group 1	This device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.	
RF Emissions CISPR 11	Class B	This device is suitable for use in all establishments including domestic, and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.	
Harmonic emissions IEC 61000-3-2	Class B		
Voltage fluctuations / flicker emissions	Complies		
Guidance and Manufacturer's Declaration – Electromagnetic Immunity			
This device is intended for use in the electromagnetic environment specified below. The customer or user of this device should assure that it is used in such an environment.			
Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment - Guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6kV contact ±8kV air	Complies	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%
Radiated RF IEC 61000-4-3	3 V/m 80MHz to 2.5GHz	Complies	Field strengths outside the shielded location from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than 3 V/m. Interference may occur in the vicinity of equipment marked with the following symbol: 
Conducted RF IEC 61000-4-6	3 Vrms 150kHz to 80MHz	Complies	
Electrical fast transient IEC 61000-4-4	±2kV power line ±1kV I/O lines	Complies	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1kV differential ±2kV common	Complies	
Power frequency magnetic field IEC 61000-4-8	3 A/m	Complies	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Voltage dips, short interrupts and voltage variations on power supply input lines IEC 61000-4-11	>95% dip 0.5 cycle 60% dip 5 cycles 70% dip 25 cycles 95% dip 5 secs.	Complies	Mains power quality should be that of a typical commercial or hospital environment. If the user of this device requires continued operation during power mains interruptions, it is recommended that the device be powered from an uninterruptible power supply or battery.

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DeVilbiss iGo® is a registered trademark of DeVilbiss Healthcare LLC.
© 2014 DeVilbiss Healthcare LLC. 01.14 All Rights Reserved.
DeVilbiss Healthcare Ltd
DeVilbiss Healthcare Pty. Limited
DeVilbiss Healthcare SAS
DeVilbiss Healthcare GmbH